
Important Before You Start: THIS IS AN ACCESSORY “QUICK SET-UP” GUIDE ONLY

Please refer to the *Dabir Operation / Service Technical Manual* for full System set-up and usage instructions. Failure to carefully read and follow all instructions and safety information may result in decreased product performance or injury.


Battery Insertion:

1. Remove battery from packaging.
2. Insert bottom of Battery into the Controller pocket, then pivot top of battery inwards until an audible “CLICK” occurs, confirming proper connection.
3. The Battery icon will appear on the LCD when properly connected. If icon is not present, Battery is NOT properly connected and Step 2 MUST be repeated.

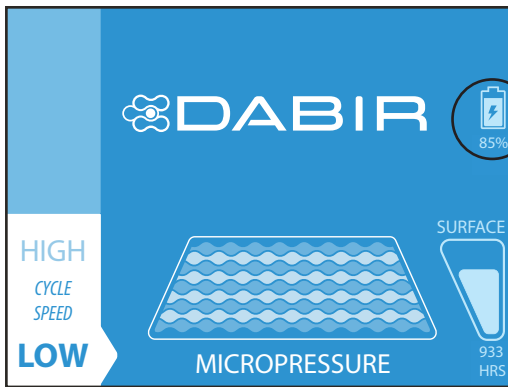Step 3: Battery Icon Check

- = Connected & Charging
- = Connected & Discharging
- = Battery Life
- = Low Charge (Yellow)
- = Battery Defective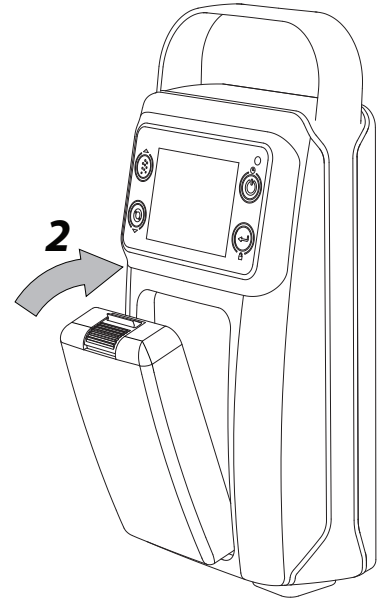

NOTE: The Battery will automatically charge when AC power is supplied to Controller.



Battery Removal:

NOTE: Controller MUST be connected to AC power source to maintain therapy after Battery removal.

4. To remove Battery from Controller, grip sides of Battery and press Orange button downward.
5. Rotate Battery outward and remove as shown.

Checking Battery Charge Level:

6. Activate Charge Level LEDs by pressing “PUSH TO TEST”.

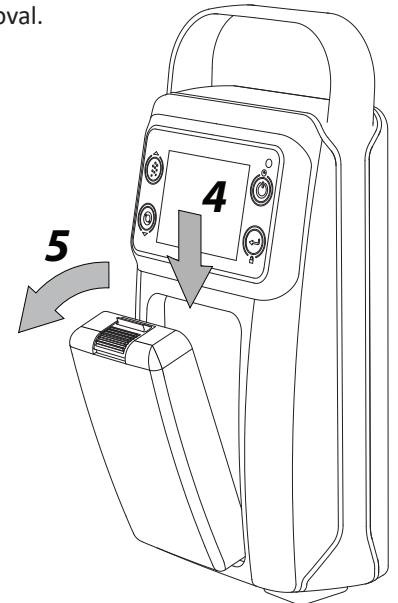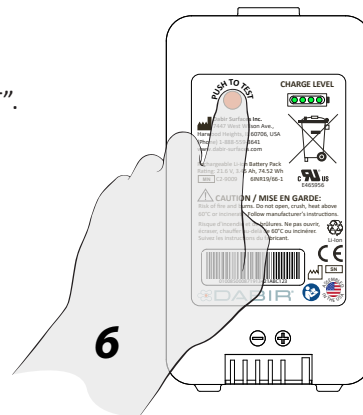NOTE:

It is the responsibility of the user to properly clean and disinfect the System prior to patient use.

Cleaning Procedure: www.dabir-surfaces.com/IFU

Approved Cleaners: www.dabir-surfaces.com/cleaners

Corporate Headquarters, Sales & Manufacturing:

Dabir Surfaces, Inc.
 Sales Inquiries:
 Technical Support:
 Authorized European Representative:
 PATENT:
 DOC# R02-0006-00121-AA

7447 West Wilson Ave., Harwood Heights, IL 60706, USA, www.dabir-surfaces.com
sales@dabir-surfaces.com, Tel: +1(888)559-3641
support@dabir-surfaces.com, Tel: +1(888)559-3642
 Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands
www.dabir-surfaces.com/patents

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