

## Installation Guide: C2-9001 (Mount - IV Pole Kit)


### Important Before You Start: THIS IS AN ACCESSORY “QUICK SET-UP” GUIDE ONLY

Please refer to the *Dabir Operation / Service Technical Manual* for full System set-up and usage instructions. Failure to carefully read and follow all instructions and safety information may result in decreased product performance or injury.

### Assembly Steps:

1. Remove components from their individual packaging containers.
2. Place Controller with rubber feet facing up on a flat, stable surface, away from any edges to prevent fall damage.
3. Use the supplied T-handle driver and Screws (2) to fasten the Accessory Mount to the Controller. (**DO NOT OVERTIGHTEN**)

**OPTIONAL:** The Accessory Mount can be assembled with either “left” or “right” handed clamp orientation. (User Preference)

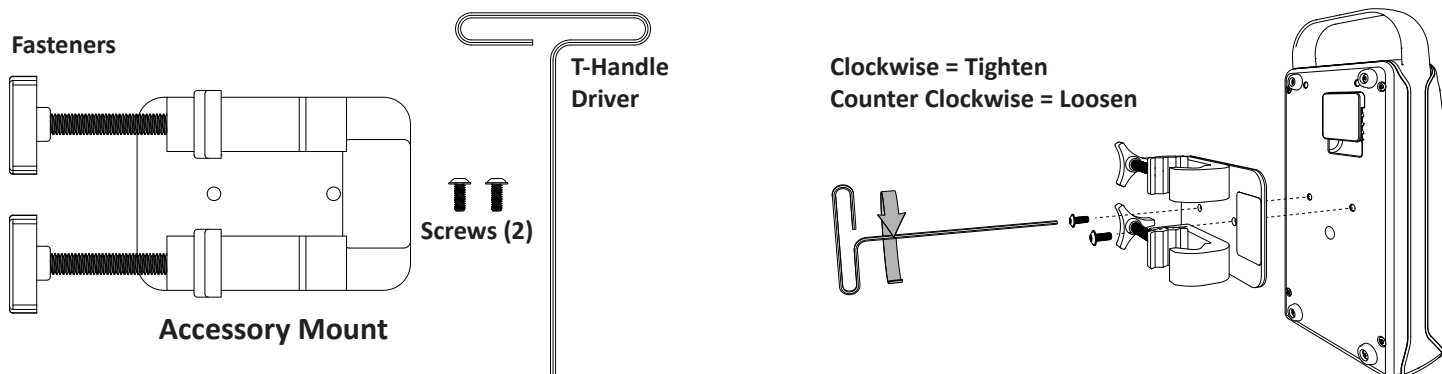

**Contents:** Model C2-9001 IV Pole Mounting Kit


**STEP 3:** Assemble to Controller (2 screws)

### Controller Installation: Attachment to customer IV Pole (NOT included)

4. Ensure Controller is powered “OFF” with the Surface and Power Cord disconnected.
5. Secure the IV Pole (if needed) to prevent it from rolling away during installation.
6. Grip the controller handle and position it against the pole at the desired install height.
7. With pole centered between the clamps as shown, tighten both fasteners to secure the Controller. (Clockwise Turn = Tighten / Counter Clockwise Turn = Loosen)
8. Use a slight pull downward to ensure the Controller is secure.

#### NOTE:

It is the responsibility of the user to properly clean and disinfect the System prior to patient use. (See Section 7 Cleaning Procedure of the *Dabir Operation / Service Technical Manual* for a list of approved / unapproved cleaners.)

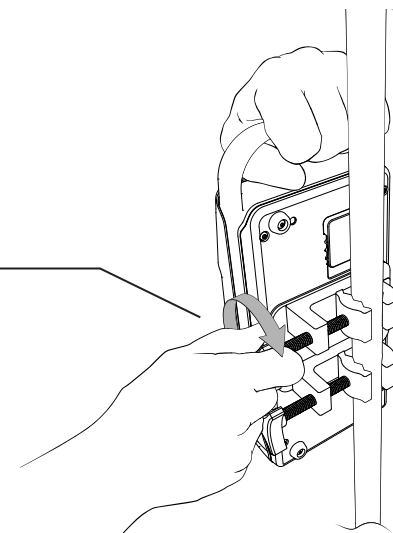

**STEP 7:** Tighten to secure Controller

#### Corporate Headquarters, Sales & Manufacturing:

Dabir Surfaces, Inc., 7447 West Wilson Ave., Harwood Heights, IL 60706, USA, [www.dabir-surfaces.com](http://www.dabir-surfaces.com)

**Sales Inquiries:** [sales@dabir-surfaces.com](mailto:sales@dabir-surfaces.com), Tel: +1(888)559-3641

**Technical Support:** [support@dabir-surfaces.com](mailto:support@dabir-surfaces.com), Tel: +1(888)559-3642

**PATENT:** [www.dabir-surfaces.com/patents](http://www.dabir-surfaces.com/patents)

**Distributor (North America):** STERIS Corporation, Customer Service, Tel: +1(800)548-4873, Fax: +1(440)639-4450; Service Support, Tel: +1(800)333-8828

**Authorized European Rep:** Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

